FastProtect Malleo

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EN – INSTRUCTIONS FOR USE

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DA – BRUGSANVISNING

SV – BRUKSANVISNING

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NO – BRUKSANVISNING

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HOW TO CORRECTLY PUT ON YOUR FASTPROTECT MALLEO

- 1 Open the touch fasteners on the strap.
- 2 Put on the FastProtect Malleo in such a way that the adaptor sits laterally over your outer ankle.
- 3 Now close the strap, enclosing your foot tightly, without exerting excessive, painful pressure.
- 4 If you can see at least 5 mm of the adaptor (pre-stressed), your FastProtect Malleo is in the right position.

USEFUL INFORMATION ABOUT YOUR FASTPROTECT MALLEO

General information

Please make sure you read the following information before you use your FastProtect Malleo for the first time, and keep it while you are using your product.

Intended use (intended purpose)

FastProtect Malleo offers you with support during all kinds of sports that put a lot of strain on the ankle. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

When should I wear FastProtect Malleo? (Indications)

- = Chronic lateral collateral ligament instabilities
- = Following ankle surgery, after painless healing
- = Degenerative joint disorders
- = Irritation of the ankle joint

When should I not wear FastProtect Malleo? (Contraindications)

With the following conditions FastProtect Malleo should only be worn in consultation with your doctor:

- = Lymph drainage disorders, arterial circulation problems, post-thrombotic conditions
- = Skin diseases or skin irritations; open wounds in the treatment area should be covered.
- = Skin irritations
- = Intolerance to the materials used for your FastProtect Malleo
- = Severe sensitivity disorders in extremities
- = Advanced peripheral neuropathy (e.g. diabetes mellitus)
- = Acute post-traumatic or post-operative treatment

Liability for failure to comply with the contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when this product is used correctly. However, if you notice any adverse changes (e.g. skin irritation) during proper use, please consult your doctor immediately. If you know that you are allergic to one or more of the product components, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please consult your doctor immediately.

Material composition

You can see detailed information printed on your FastProtect Malleo.

Washing and care instructions

Air your FastProtect Malleo after use and leave it to dry. If needed, you can wash it by hand. FastProtect Malleo is not suitable for washing machines/tumble driers. Please note the care instructions printed on the strap of the product.

Correct storage

Do not store your FastProtect Malleo with sharp or heavy objects. Do not store the product above 30 °C or below -10 °C. Please store your product in a dry place and protect it from humidity. Avoid direct sunlight and frost.

Shelf life

The product has a shelf life of two years. The usage period with normal use is one year.

Damage

If the adaptor/strap of your FastProtect Malleo is damaged, replace it with a new one. Do not try to repair it. We will not accept any liability for malicious/deliberate damage and individual changes to the FastProtect Malleo. If you have any questions, contact our sportomedix customer service team (email contact@sportomedix.com, phone: +49(0)8251 901 469).

Wear and tear

Please ensure that there are no signs of wear and tear in your trainers, for example on the inside of the heel, that may rub on the material of your FastProtect Malleo, thus damaging it.

Disposal

You can dispose of FastProtect Malleo with general household waste.

FURTHER INFORMATION

At which temperatures can I wear my FastProtect Malleo?

Temperatures should not exceed 40 °C or be below -10 °C.

Resistant

You can apply creams and ointments underneath your FastProtect Malleo. This will not damage the material.

In the event of complaints related to this product, such as damage to the material or a problem with the fit, please contact your medical retailer directly or our sportomedix Customer Service Team (email: contact@sportomedix.com, Tel.: +49(0)8251 901 469). Only serious incidents that may lead to a major deterioration in the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

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