FastProtect Malleo

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HOW TO PUT ON
FASTPROTECT MALLEO

1 Open the Velcro fastener on the belt.

2 Put the Fast-Protect Malleo on so that the adaptor sits on your external ankle.

3 Now fasten the strap so that your foot is firmly enclosed without exerting too much painful pressure.

4 If you can see at least 5 mm of the adaptor (prestressed), your FastProtect Malleo is in the right position.

FastProtect Malleo

Adaptor – The heart of your innovative FastProtect Malleo is the so-called adaptor. It works when the movement becomes too fast for your ankle joint. The body needs about 200 milliseconds (ms) to react to an unhealthy movement. An injury normally occurs between 0 and 200 ms. The adaptor reacts three times as fast as the human body and already intervenes after 20 - 30 ms. The full force is already deployed after 50 ms. The foot remains free to move and the FastProtect Malleo only has a high protective effect when it is needed.

Bandage – Your FastProtect Malleo is made of an exclusive fabric specially developed for sports textiles. This fabric offers a high compression performance with exceptional freedom of movement at the same time.

ALL YOU NEED TO KNOW ABOUT YOUR FASTPROTECT MALLEO

General information

Please read the following information before using your FastProtect Malleo for the first time and keep it for the duration of use.

Intended use (Purpose)

Our FastProtect Malleo offers you support for all kinds of sports that put a heavy strain on the ankle joint. The manufacturer is not liable for damages / injuries caused by improper handling or misuse.

For which discomforts should I wear the FastProtect Malleo? (Indications)

- = Chronic external ligament instabilities
- = After surgical interventions on the ankle joint, after painless healing
- = Degenerative joint diseases
- = Irritation of the ankle joint

When should I not wear FastProtect Malleo? (Contraindications)

The FastProtect Malleo should only be worn in consultation with a doctor in case of the following symptoms:

- = Lymph drainage disorders, arterial circulatory disorders, postthrombotic conditions
- = Skin diseases;
Open wounds in the section of the body being treated must be covered.
- = Skin irritations
- = Intolerance to the materials used for your FastProtect Malleo
- = Severe sensitivity disorders in extremities

- = Advanced peripheral neuropathy (e. g. diabetes mellitus)
- = Acute post-traumatic or postoperative treatment

In the event of non-compliance with the contraindications, no liability can be assumed by the Julius Zorn GmbH group of companies.

Side effects

When used properly, no side effects are known. However, if negative changes (e.g. skin irritations) occur during proper use, please contact your doctor immediately. If an intolerance to one or more ingredients of this product is known, please consult your doctor before use. If your discomfort worsens during wear, consult your doctor immediately.

Material composition

Please refer to the imprint on your FastProtect Malleo for the exact details.

Proper storage

Do not store your FastProtect Malleo with sharp, heavy objects and do not store above 30 °C and under -10 °C. Store in a dry place and protect from moisture. Avoid direct sunlight and danger of frost.

Washing and care instructions

After use, you should air and dry your FastProtect Malleo well. If necessary, you can wash it by hand. The FastProtect Malleo is not suitable for the washing machine / dryer. Please follow the care instructions on the imprint on the belt of the product.

HAND
WASH

DO NOT
DRY-CLEAN

DO NOT
BLEACH

DO NOT DRY
IN A DRYER

DO NOT
IRON

Shelf life

The product has a two-year shelf life. The usage period in normal use is one year.

Damage

If the adaptor / tape of your FastProtect Malleo is damaged, replace it with a new one and don't try to repair it. We assume no liability for intentional / deliberate damage and individual changes to FastProtect Malleo. If you have any questions, please contact our sportomedix customer service (E-mail contact@sportomedix.com, Phone +49(0)8251 901 469).

Wearout

Make sure that your training shoes do not show any wear, e.g. on the inside of the heel, which could rub against the material of your FastProtect Malleo and thus damage it.

Disposal

The Adaptor may only be disposed of with household waste if it is 100% incinerated. You can dispose of your FastProtect Malleo with the conventional household waste.

FURTHER INFORMATION

At what outside temperatures can I wear my FastProtect Malleo without any problems?

The outdoor temperature should not exceed 40 °C and not be less than -10 °C.

Resistant

You can apply creams and ointments under your FastProtect Malleo. The material will not be damaged.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the retailer or our sportomedix customer service (E-mail contact@sportomedix.com, Phone +49(0)8251 901 469). Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

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